D₂₀

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Viruton® Bohrs

Powerful concentrate for cleaning and disinfecting rotary instruments

Application:

Ready to use formulation intended for simultaneous cleaning and disinfection of precision rotary instruments and other medical instruments. It is recommended for disinfection: drills, bone cutters made of steel, hard metals, diamonds, tools, abrasives and polishing agents such as silicone rubber, ceramic tile and tools for root canal treatment. Corrosion inhibitors, making ideal for all of the instruments and stainless steel alloys. It has a pleasant floral scent. It can be applied by immersion in an ultrasonic bath.

Applications references:

Instruments immediately after use immerse in undiluted preparation. Carefully submerge all items. After a certain time remove from tank or washers, rinse in water and immediately dry with a disposable napkin. In case of visible pollutants remove them with a brush and disinfect product again in preparation. Do not mix with other detergents or disinfectants. Do not use on aluminium

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DZ.1101.01 1 litre bottle / Butelka 1 l z dozownikiem

Viruton® Forte

Concentrate for cleaning and disinfecting surgical instruments

Application:

Viruton Forte is a very effective preparation for simultaneous cleaning and disinfection of all kinds of surgical instrument (except for flexible endoscopes). Viruton Forte has a broad spectrum in low use concentrations. The use of corrosion inhibitors ensures high material compatibility with instruments made of stainless steel and aluminium, medical devices and laboratory glass, porcelain, metal, rubber and plastics.

Applications references:

The concentrate must be diluted before use with water to the desired concentration for the specified table. 1 liter bottle includes an integrated dispenser: 10 ml at 1 liters of water gives a concentration of 1%. Instruments immediately after use immerse in the solution. All parts and holes must be completely filled with the solution. Before the first use of the preparation instruments should be washed with clean water so that it does not come into contact with other agents.

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Viruton® Extra

Powerful concentrate for cleaning and disinfecting medical instruments

Application:

Viruton Extra is a highly effective preparation for simultaneous cleaning and disinfection of all kinds of thermo labile and thermo stable surgical instrument (except for flexible endoscopes). Modern formula provides excellent cleaning effects even at high protein load in a short duration of action. Viruton Extra has a broad spectrum in very low use concentrations. The use of corrosion inhibitors ensures high material compatibility with instruments made of stainless steel and aluminium, medical devices and laboratory glass, porcelain, metal, rubber and plastics.

Applications references:

The concentrate must be diluted before use with water to the desired concentration for the specified table. 1 liter bottle includes an integrated dispenser: 5 ml at 1 liters of water gives a concentration of 0,5%. Instruments immediately after use immerse in the solution. All parts and holes must be completely filled with the solution. Before the first use of the preparation instruments should be washed with clean water so that it does not come into contact with other agents.

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Viruton® Strong

Concentrate for cleaning and disinfection

Applications references:

Viruton Strong is a high-end Solution for disinfection and cleaning instruments with a new active substance, which is characterized by a short contact time and use in low concentrations. This product has great cleaning properties, with broad spectrum of biocide action. It is aldehyde- and phenol free and contains corrosion inhibitors for your instruments safety. It is very suitable for disinfecting laboratory utensils and medical instruments of glass, porcelain, metal, rubber, plastics. Concentrate should be diluted with cold clean water to obtain the desired concentration. The instruments must be immersed in the solution immediately after use. All surfaces and cavities of instruments should be covered completely with working solution. After expiry time of disinfection, instruments must be rinsed with water and dried. Working solution is stable and active 14 days.

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Velox® Top AF

Ready-to-use formulation for cleaning and disinfection of surfaces of medical equipment

Velox Top AF is a ready-to-use formulation with a wide spectrum in microbiological activity combined with short contact times as well as highly effective cleaning and material preserving properties.

Applications references:

This ready-to-use rapid disinfection liquid shall be sprayed from a distance of 20-30 cm onto the surfaces or objects to be disinfected and evenly spread with a lintfree, uncolored disposable cloth. Ensure that the disinfected surfaces are completely moistened. After application, allow the disinfectant for 15 minutes to take effect or in accordance with the exposure times listed below. In the event of strongly contaminated surfaces, use additional cleaner, but do not mix with other cleaning agents or disinfectants.

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Velox® Wipes

Alcohol-based-formulation wipes for cleaning and disinfection of surfaces of medical equipment

Presaturated with an alcohol-based formulation, Velox Wipes is a cleaner and disinfectant in one. Dedicated for rapid disinfection of small surfaces of medical equipment, chairs, beds, dental ends before the sterilisation process and prosthetic spoons and silicon moulds. Also recommended for surfaces having contact with food. Free from aldehyde and phenol, does not discolour disinfected surfaces. It has as a pleasant grapefruit-and-mandarin scent. Tested and recommended by Famed - a leading European producer of top quality hospital equipment.

Applications references:

Take a tissue from the tub and wipe a surface to be cleaned and disinfected. Close the lid. After opening the packaging, use the tissues within 21 days. Do not use if a tissue is dry. Do not use on sensitive to alcohols areas (e.g. acrylic glass). The product is intended for use by professional users.

Active ingredients: amine, quaternary ammonium compounds

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AHD 1000 Sterisol

Alcohol gel to surgical and hygienic hand disinfection . Microbiologically clean the entire period of use . It does not contain preservatives, and dyers.

Gel formula allows for precise spreading and rubbing the product into the skin
Exhibits an immediate and prolonged effect confirmed by tests
Has a broad spectrum biocides - fights bacteria , mycobacteria , fungi and viruses
Exhibits effective action even in the presence of organic pollutants
Manufactured under aseptic conditions
Microbiological purity of the product throughout the period of use is guaranteed
A special valve - the patented valve system to prevent suction of air into the bag along with microorganisms
S pipe - valve has created a product that does not contain bacteriostatic substances and preservatives , which are often the cause of allergies and skin irritations
Recommended for frequent use for people with very sensitive skin and tendency to allergies and prevent dry skin - contains moisturizers
Works very gentle on the skin, leaves no chemical substances aggravating and irritating to the skin

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AHD 1000

Alcohol based hand and skin disinfectant

Control of bacteria (incl. Tb), fungi and enveloped viruses, rota virus, adeno virus
Disinfection of hands and non-injured skin
Use externally and undiluted only!
Pour at least 5 ml AHD 1000® in the palm of the hands and rub on hands and forearms until dry, giving particular attention to the fingertips and to the areas around the nails. Duration: 1 1/2 min. each.

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AniosGel 85 NPC

Alcohol-water, thixotropic gel for hygienic and surgical hand disinfection by rubbing. Ideal for people who for professional reasons often have to disinfect their hands.

- 1 Has a broad spectrum biocides - fights bacteria, mycobacteria, fungi and viruses
- 2 Inactivated polio virus in 30 seconds
- 3 Exhibits an immediate and long (3hours), confirmed by tests
- 4 Thixotropic properties facilitate the application of the preparation (not sprayed) and provide accurate distribution and rubbing the product into the skin
- 5 Hypoallergenic - dermatologically tested and does not contain allergenic ingredients, flavorings and colorings is very gentle, does not irritate or dry the skin
- 6 Has a natural pH
- 7 Recommended for frequent use for people with sensitive skin and tendency to allergies

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