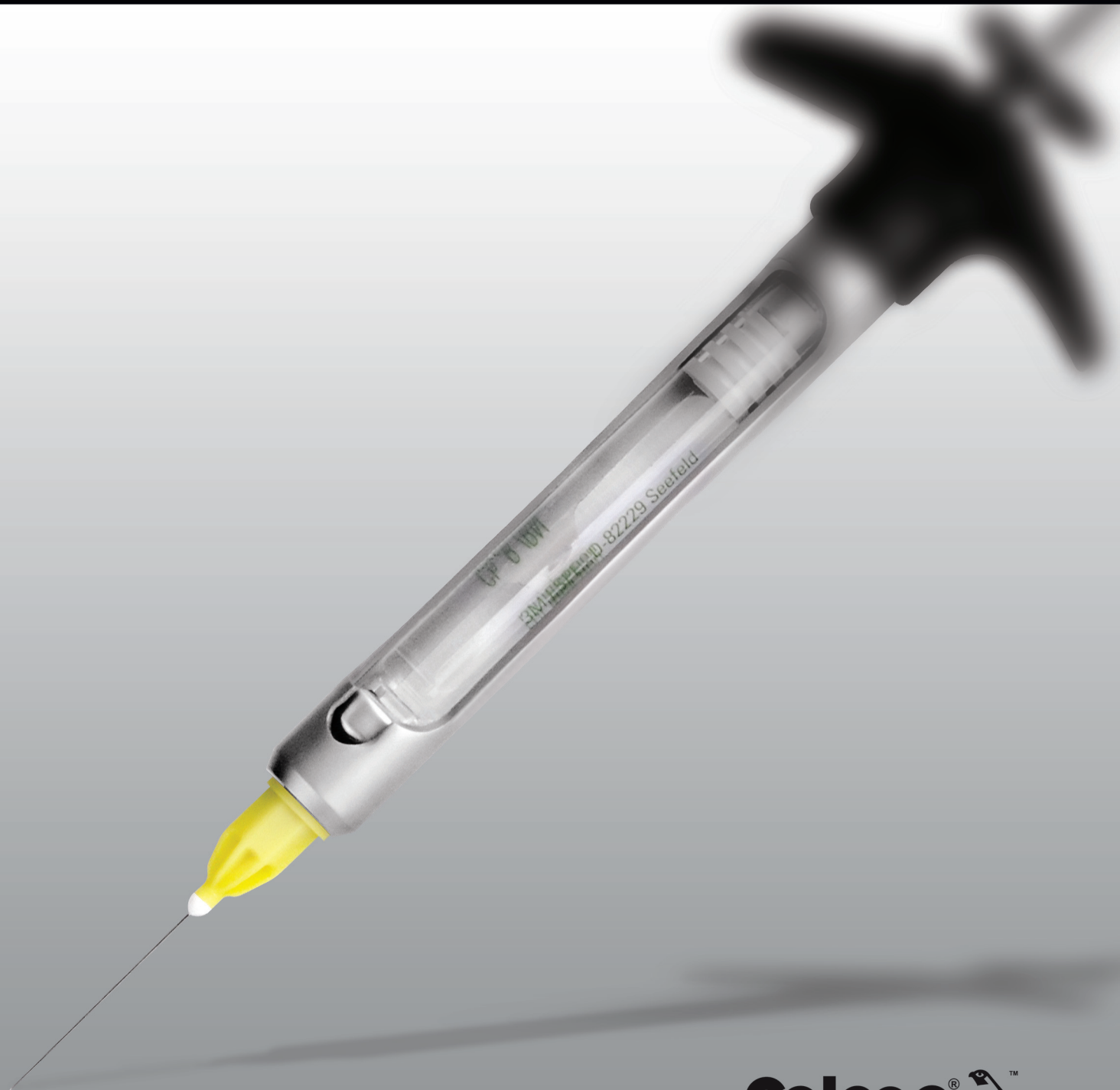
D₉

Anaesthesia / Anestezjologia

Stainless steel series
Made of 18-8 stainless steel
Longer life than Brass syringes
No chipping of chrome
Guaranteed quality
Sterilizable by any method

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
During injection, aspiration can be achieved either manually by retracting the plunger backward or simply releasing pressure on plunger will produce self aspiration.

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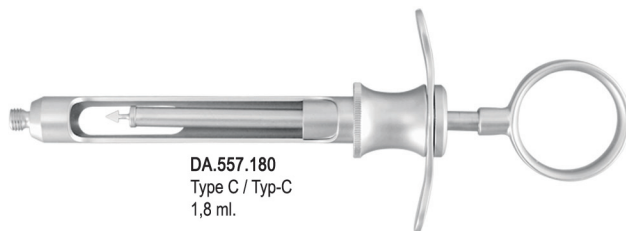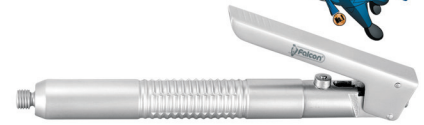
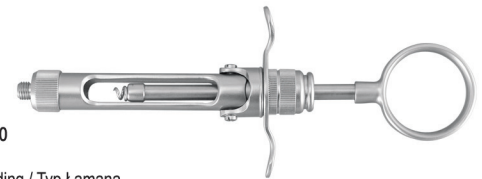
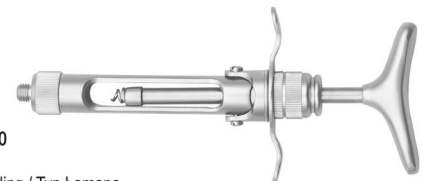
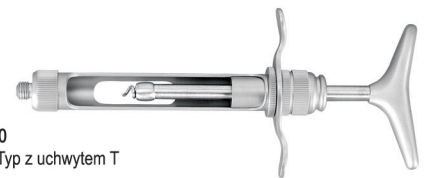
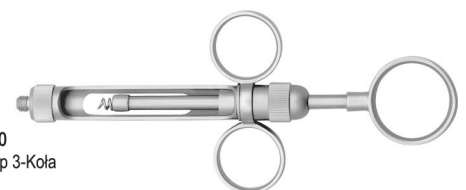
DA.595.180
Euro / Typ Euro
1,8 ml.

DA.705.180
3- Ring / Typ 3-Koła
1,8ml.

Made of Brass
Chrome plated
All Brass series syringes have stainless steel plungers
Guaranteed quality
Sterilizable by any method

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DA.095.180
Swedish Type / Typ Szwedzki
1,8ml.

DA.060.180
Pistol-Ject
1,8ml.

Injects 0.20 ml with every click
Lever action for high injection pressure

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
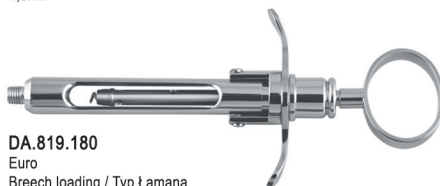
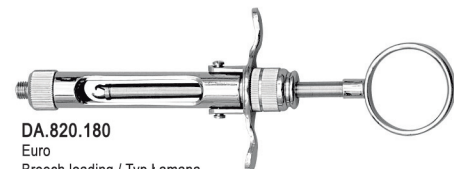
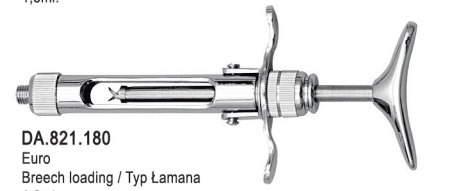
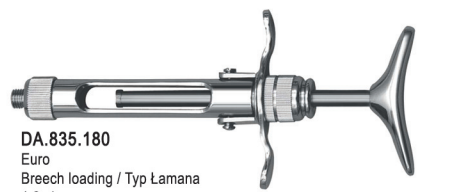
Self aspiration: How it works

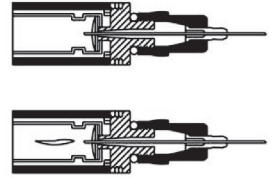During use, the thumb pressure on the plunger or thumb disc stretches the diaphragm over a small metal projection inside of the syringe nozzle, increasing pressure inside of the cartridge. As the thumb pressure is released, pressure inside the cartridge lessens and aspiration occurs automatically. (Instruction for use are supplied with the product)

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Disc system syringes

Dual self aspiration system:

Disc system syringe has dual aspiration feature to achieve self aspiration.

Aspiration before injection:

Insert syringe in tissue and press thumb disc to achieve aspiration.

Aspiration during injection:

Automatic aspiration occurs whenever pressure on the plunger is released.

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SS•2000
Stainless Steel Series

Falcon Needle Re-Cappers are the ideal way to prevent needle stick injuries when recapping needles. Place the upturned needle cap into Falcon Re-Capper and re-sheathing becomes a simple one hand procedure.

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